



# HEALFUSION

3 in 1 Total Care Solution - **Skin / Body / Scalp** by Micro Pore Fit

*Spread healing into your skin*

# WHY HEALFUSION?

Heal Fusion is a combination of a powerful water jet and a solution. This is a device that delicately fits the skin and powerfully cleans pores. Pore fit waterstream cleans pores more tightly than other devices and provides the optimal solution for penetrating effective nutrients into the skin.

*Just 10 Minutes*

Skin, Body, Scalp scaling ! Let's get started now!



**HEALUX** identity

## **Total Anti-aging Solution Company**

Healux is a company that provides total anti-aging solutions as a developer and manufacturer of skin laser equipment, beauty equipment, and 'i-Thread' lifting thread.

Healux has been promoting the excellence of thread lifting through dermatologists, plastic surgeons, doctors, and online marketing platforms in 60 countries around the world.

In the future, we plan to become a 'medical aesthetic medical device company' through medical cosmetic devices, Nd-YAG laser, and cosmetics for hospitals and clinics.

# HealFusion



## Point 01

### Solution Based Technology

4-step customized solution for cleansing, normal skin, oily skin, and brightening skin - A, B, C, Dr.1

## Point 02

### Easy to operate

- **Auto Mode** : auto mode set by operator
- **Manual Mode** : intensive care mode set for targeted area

## Point 03

### 3 in 1 Total solution care

**Apply to the face, scalp, and body** with a larger rolling tip that covers a larger skin area and with powerful motor, higher rotational force can be achieved

## Point 04

### Micro Air-water Flow System

Micro cleaning technology using a more powerful vacuum device, water jet, and powerful rotational force

## Point 05

### Treatment Collaboration

Maximizes treatment effect and pre-treatment support before getting treatment from laser and high-frequency devices



THREAD

**i-THREAD**  
Thread Codes



DEVICE

HealFusion  
Caspian 2



COSMETICS

mayglory  
MESOGLORY  
FOR DERMATOLOGICAL USE



MARKETING

Keys

# LARGER TIPS for Minimizing Treatment Time

Micro water jet combined with  
Special vacuum device




## 1 SKIN TIP

### Step. 1 Cleansing tip - Loop type

Removal of dead skin cells  
and impurities

Loop-type tip specialized for exfoliating  
by gently suctioning the skin

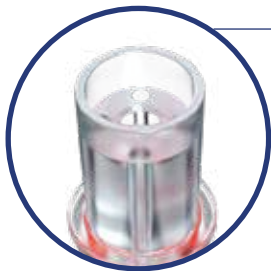

## 2 PORE TIP

### Step. 2 Peeling & Boosting tip

**Solution penetration type**

Penetrating type tip specialized  
for intensive nutrition supply  
with a fine stream of water.



## 3 SCALP TIP

### Tips for scalp care

The first hair type tip  
specialized for the hairline.  
Scalp customized tip  
to supply scalp nutrition



Customized for each skin type  
for smooth peeling

## PORE FIT Solution



*step 1*



or



*step 2*



*step 3*

### A+ Cleansing water peeling

Cleansing skin,  
Removing dead cells

Micellar particles + powerful water stream removes large impurities and surface sebum

250 ml / 8.45 fl.oz.

### B+ PHA peeling

Daily pore scaling

PHA peeling that gently penetrates  
Moisture replenishment solution for moisture barrier

250 ml / 8.45 fl.oz.

### C+ LHA peeling

Intensive pore scaling

LHA ingredient targeting problematic pore  
Moisture replenishment solution for moisture barrier

250 ml / 8.45 fl.oz.

### Dr. Brightening solution

Brightening pore  
& elasticity care

Brightening care that brightens skin tone of skin marks caused by pore


250 ml / 8.45 fl.oz.

\* Scalp solution will be updated by 1st quarter of 2024

# • Stages for Healfusion treatment

## Basic course for face 10 min


\* Using proper tip per stage is a must

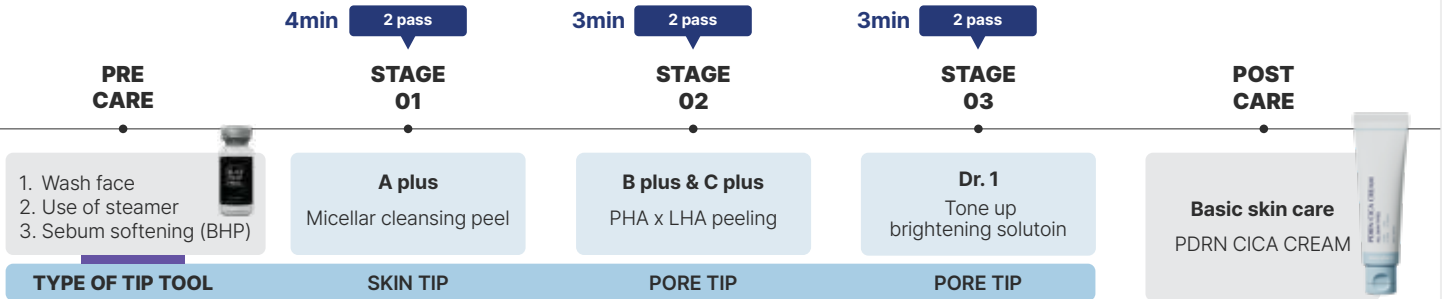

## Basic course for body 20 min


(ideal for chest and back acne)

\* Using proper tip per stage is a must

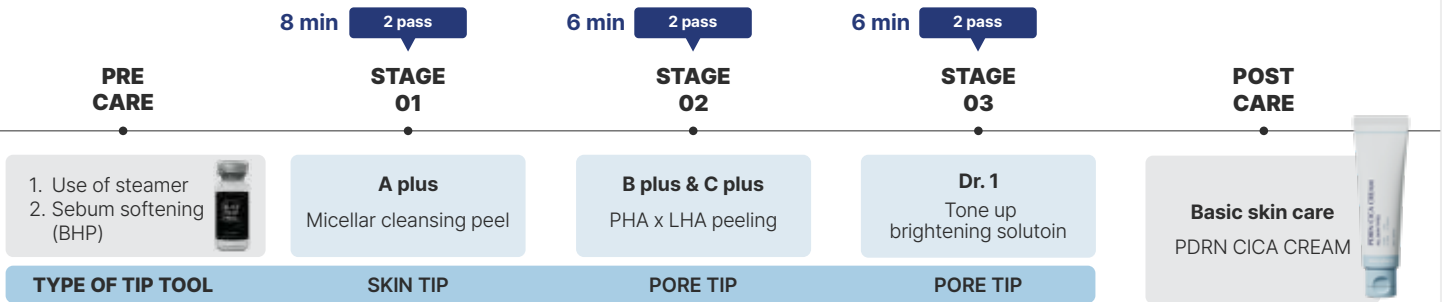

\* Scalp solution will be updated by 1st quarter of 2024

# • Types of solutions used for each area

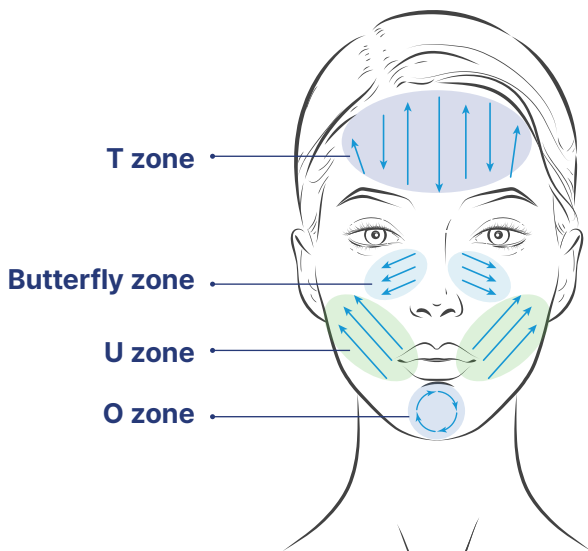

- T zone**  
A, B, C, D solutions for intensive care due to excessive sebum secretion on this area
- Butterfly zone**  
B, C, D solutions for full care due to this area is more dry than other areas
- U zone**  
B, C, D solutions for intensive full care on black head
- O zone**  
A, B, C+ solutions for intensive care due to excessive sebum secretion on this area

*Spread healing into your skin*

## BEFORE & AFTER

- Improved skin tone



BEFORE



AFTER

- Removed impurities



BEFORE



AFTER

- Removed black head / white head



BEFORE



AFTER




BEFORE



AFTER



HEALUX HomePage

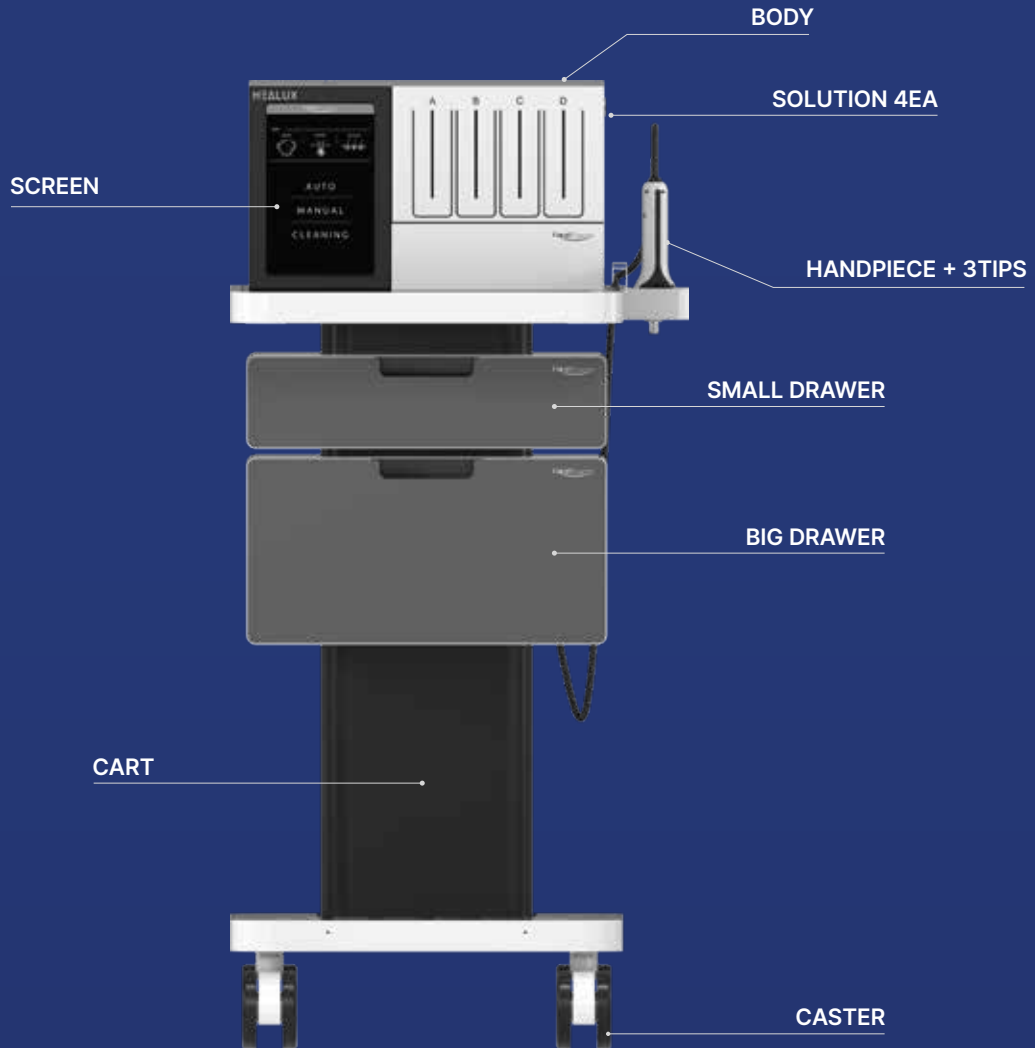

## SPECIFICATION

POWER	220V ~ 60Hz
OUTPUT	VOLTAGE 24v * 1A
PRODUCT SIZE	<b>Body</b> 455 mm X 260 mm X 370 mm <b>Cart</b> 600 mm X 970 mm x 570 mm
WEIGHT	42kg
SOLUTION BOTTLE	250ml*4EA
OPERATION SCREEN	135 mm X 215 mm
ACCESSORY	Handpiece Cleaning cap Tip holder TIP [ Skin Tip / Pore Tip / Scalp Tip ] Solution A,B,C,Dr.1 + α Cleaning bottle, Suction bottle Foot switch